


Specimen collection instructions

Ordering and Shipping

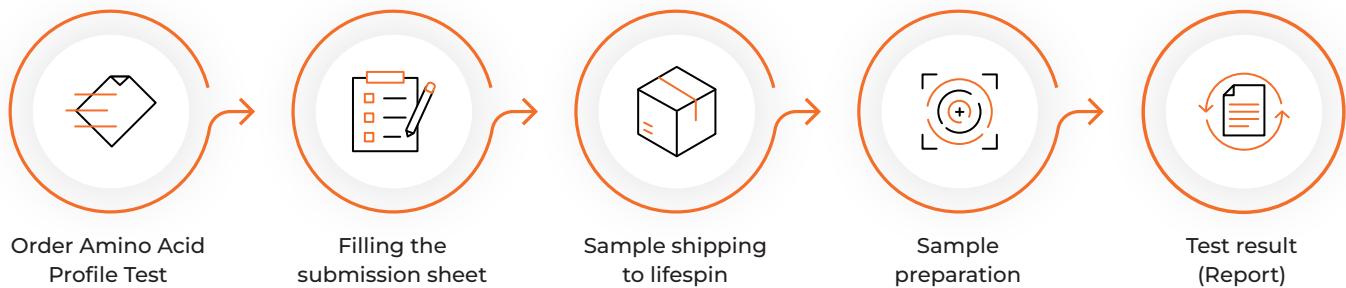

The commissioning of the examination and the shipment (optional dry ice) will be organized in consultation with lifespın.



customerservice@lifespın.health



+49 941 942 898 22



Please add submission sheet to the package

Important notes

The patient must be at least 18 years old and does not need any pre-treatment. The submission sheet must be filled in completely.



Sample Type:
Serum



Sample volume:
min. 400 µl Serum

Sample identification

The Sample of Serum must be assigned an unique identification number.



This must be visible on all documents sent:

- On each page
- On all sent tubes
- No use of clear names

Specimen collection instructions

Specimen Collection


Blood collection for serum collection is performed in dedicated, commercially available CE-marked blood collection tubes and must comply with the recommendations of the D/A/CH/FL Working Group on Extraanalytical Quality.




The extraction of the serum is collected according to the manufacturer's instructions for the monovettes (standard protocol).




Please wait for complete coagulation (clotting) of the sample (at least 30 min at room temperature).

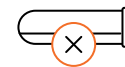

Lying

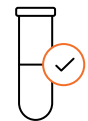

Standing

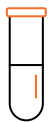

Transfer at least 400 µl of the serum into a tube

- Tightly sealable
- Dry ice resistant
- CE-marked

Sample of serum must be sent to lifespın as soon as possible




Storage the sample of serum up to 6h at +8°C



If storage exceeds the maximum duration of 6h, storage at -20°C



- Heat treated**
(e.g. Interruption of cold chain)
- Contaminated**
(e.g. with disinfectant)

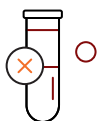

Hemolytic



Lipemic



Icteric



Take a new sample